

"VELPEAU" SLING



HOW TO PLACE



Ref: FI-500

Recommended to fasten the arm after a
PACEMAKER implantation



The sling is made with resin free hypoallergenic materials welded using an ultrasonic process and it is produced in a single size.

None of our products contain foam, as foam generates heat and causes sweating and skin irritation.



Pass the arm through the belt until it reaches the biceps.



Wrap around the body.



Pass through the ring and fasten with VELCRO. Place the arm between the body and the sling.

Distributor



2mil artesanía sanitaria, s.l

✉: c/Amadeo Vives nº 8 - 28290 LAS MATAS (Madrid) SPAIN

☎ & FAX: +34-916309208 / 1254

ER: +34-606957883

E-Mail: 2mil@2mil.com

www.2mil.com