

# "VELPEAU" SLING

(product composed of two pieces)



Ref: FI-505 XL  
FI-505 L  
FI-505 M  
FI-505 S

The sling is made with resin free hypoallergenic materials welded using an ultrasonic process and it is produced in a single size.



As result we obtain a very light and effective with two slings and with only two velcro fasteners.  
None of our products contain foam, as foam generates heat and causes sweating and skin irritation.



## HOW TO PLACE



Pass the arm through the belt until it reaches the biceps.



Wrap around the body, pass through the ring and fasten with velcro.



Introduce the hand into the second piece as shown. Pass the end between the body and the first piece and wrap around the back.

**2mil** artesanía sanitaria, s.l

✉: c/Amadeo Vives nº 8 - 28290 LAS MATAS (Madrid) SPAIN  
☎ & FAX: +34-916309208 / 1254 ER: +34-606957883  
E-Mail: 2mil@2mil.com [www.2mil.com](http://www.2mil.com)



Distributor